Artificial Teeth

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EN Artificial Teeth. Instructions for use

Product description

These are dental pieces (anterior and posterior teeth) manufactured in series. They are known as artificial teeth made of synthetic resin (polymethylmethacrylate) or acrylic teeth. As stated in the literature, polymethylmethacrylate has chemical properties suitable for use in synthetic teeth. It is non-toxic, insoluble in oral fluids and has acceptable mechanical properties of compressive strength, abrasion resistance and resilience.

Composition Acrylic copolymer based on PMMA > 99% / Inert mineral pigments and other non-toxic substances < 1%. Intended use Unidesa Odi artificial teeth are designed for use as components of dentures and other dental restorations customised to the patient's mouth.

Intended users Dental health professionals, dental technicians and dentists, exclusively.

Indications Complete and partial dentures, removable and fixed / Temporary and skeletal restorations.

Contraindications Patients with known allergic reaction/sensitivity to any of its chemical components.

Instructions for use

Use the mould chart (shape chart) to select the shape and size of the teeth that best suit the patient's mouth so consider the indicated measurements and recommended articulations among the different models. Use the shade guide for simple but accurate shade determination. The shade guide presents several groups of different shades corresponding to the dominant colour shades of natural teeth: These groups appear in various intensities thus covering the entire spectrum of shades found in natural teeth. Hold the shade guide close to the patient's mouth and select the dominant shade group. Determine the exact shade by extracting all groups of colour chips from the shade guide and comparing the intensity. It is important to assess the first choice so that the retinal cones of the eye are exhausted after a few seconds of intense colour determination and may generate doubts in the choice. If doubts arise, a glance at an area of neutral colours will help to rest the eye and, after waiting a few minutes, return to colour determination.

Methods to follow during the repair and heating process

To maintain the integrity of the teeth, several steps should be followed in the preparation of the wax-up and flask: a) Flame brush the wax-up with caution. Avoid point flame directly against the teeth. High intensity heat can burn the surface of the acrylic teeth. b) After completion of the wax-up, all wax residues should be removed from the exposed tooth surface. We recommend cleaning the surface of the teeth with hot water before investing. Any traces of wax remaining on the surface of the teeth will result in a fine indentation between the veneer and the acrylic teeth once the case has been boiled. To ensure optimum adhesion to the denture base resin, remove the wax and prime the tooth base with acrylic monomer. When packing the case with denture base resin, excess monomer could seep into the crevice and contaminate the investment. This is the main reason for tooth wear and difficulty in polishing plastic teeth. The time spent to thoroughly clean the teeth prior to investment is well rewarded. c) This method also applies to the repair of acrylic teeth by the heat method or when using the automatic polymerisation process or fast curing materials. Refresh the tooth surface by grinding with a bur after carefully removing the wax when using cold curing materials. This step will ensure a good bond with the self-curing materials.

Precautions

The product is not resistant to oxidising chemicals and strong acids. It is recommended to use a suitable air filtering mask to avoid inhalation of dust particles during polishing or grinding. The material is not visible by X-ray. Do not reuse the product. Consider the physical properties of the material to be used. Once the prosthesis has been made, clean the artificial teeth by removing dust and plaster residues with a suitable cleaner and rinse them by immersing them in water.

Interactions

Potential interactions with other products used in the treatment are highly unlikely because the product is a fully polymerised acrylic resin-based material. Before using the product, the dentist should consider known interactions and cross-reactions of the product with other materials already present in the patient's oral environment to ensure that the restoration can be placed safely and durably.

Side effects In individual cases, rashes, sensitivity or allergies may occur when using the acrylic resin-based prosthesis, in which case do not use it and consult a doctor.

Risks of reuse Caution: reusing used dentures and other dental structures on other persons may cause infections and transmit diseases.

Storage conditions Store artificial teeth at room temperature in a clean, dry place away from sunlight.

Disposal of the product

Unidesa Odi artificial teeth do not require specific disposal and can be disposed of in household waste. The presentation elements accompanying the teeth, such as the support splints and the boxes, should be disposed of in their corresponding containers (plastic and cardboard respectively). Contaminated products that carry biological and/or infection risks should be disposed of in accordance with the applicable regulations.

Information for the patient

The practitioner should inform the end-user of the following indications: a) Normal oral hygiene should be maintained. Instruct the patient to adopt daily hygiene protocols*, suggest to avoid excessive brushing which may cause abrasive wear, and to avoid immersion in alkalis, acids or other substances that may damage acrylic materials. b) The material is not visible to X-rays. c) Possible material wear due to grinding of teeth, strong bites (avoidance) or abnormal antagonists. d) In case of allergic reactions to the material, consult a doctor. e) Of the following residual risks: Allergies and irritations caused by the material. Breakage and possible ingestion of the product. Deterioration of the product. Incorrect use of the product. Inadequate hygiene and maintenance of the product.

"Daily hygiene protocol". Patients should brush their artificial teeth at least once a day; dentures should be removed and gently cleaned daily. Soak and brush with a soft bristle brush and a non-abrasive denture cleanser to remove food, plaque and other deposits.

Manufacturer's responsibility

Material for dental use: The manufacturer is not responsible for any damage that may result from use other than in accordance with the instructions for use. Class IIa medical device: Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the incident occurred.

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Union Dental, S.A.