# idobase High Impact

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## EN Idobase High Impact acrylic resin. Instructions for use

### Product description

UNIDESNA ODI presents IDOBASE HIGH IMPACT resin, especially indicated for the manufacture of removable dental prostheses with high impact resistance and by means of the traditional hot curing technique. This is a resin from the IDO series, with a formulation that allows excellent results to be obtained by working in an easy and practical way. It is available in pink and pink veined colours and stands out for its formidable performance in accordance with ISO 20795-1: plasticity, colour fastness, waterproofing, adhesion to artificial teeth, etc. Idobase HI resin is cadmium-free.

### Composition

Idobase HI powder: Reinforced PMMA based copolymer > 99% / Catalysts and pigments < 1%.

Idobase HI liquid: Methyl methacrylate > 90% / Dimethacrylate < 10%.

### Intended use

Idobase HI resin enables the fabrication of removable dental prostheses with high impact resistance.

**Intended users** Dental health professionals, dental technicians and dentists, exclusively.

**Indications** Removable dental prostheses.

**Contraindications** Fixed dental prostheses in the mouth for an uninterrupted period of more than one month. Patients with known allergic reaction to any of its chemical components.

### Instructions for use

**Dosage:** Ratio 21 g powder : 10 ml liquid Idobase HI

Mix at room temperature in a mixing cup and allow to stand for 30 seconds. Whisk vigorously using a metal spatula, leave the mixture to stand for 2 minutes (with the container covered to prevent evaporation) and whisk again vigorously. Do not add more liquid even if it seems necessary. After resting for another 10 minutes the dough is ready to be packed for another 15 - 20 minutes as long as the temperature is kept at about 23°C.

**Rapid curing:** Immerse the muffle in cold water, heat the water to about 70°C for 20 minutes, bring to the boil and keep for one and a half hours. Remove the muffle and leave to cool in the air for 20 minutes, then immerse in cold water for a further 15 minutes before opening.

**Standard curing:** Immerse the muffle in cold water, heat the water to about 70° for 6 hours, bring to the boil and hold for two hours. Let the muffle cool in the water to room temperature before opening.

**Moulding:** For moulding, use materials such as plaster or hard plaster and conventional methods for the compression moulding technique.

**Finishing and polishing:** Idobase HI can be finished and polished by the usual method.

**Repairs and undercuts:** Unidesna Odi's self-curing resins (Idopress and Idofast) can be used for relining and denture repairs.

### Precautions

The product is not resistant to oxidising chemicals and strong acids. During finishing, use a hoover and/or mask

to avoid breathing dust. The material is not visible with X-rays. Do not reuse the product and do not use expired materials. Do not vary temperatures and working parameters established by the manufacturer (mechanical modifications of the material could occur, with consequences for the final manufactured product). Do not contaminate the product during the processing phase.

### Interactions

Potential interactions with other products used in the treatment are very unlikely because the product is a fully polymerised acrylic resin-based material. Before using the product, the dentist should consider known interactions and cross-reactions of the product with other materials already present in the patient's oral environment to ensure that the restoration can be placed safely and durably.

**Side effects** In individual cases, rashes, sensitivity or allergies may occur when using the acrylic resin-based prosthesis, in which case do not use it and consult a doctor.

**Risks of reuse** Caution; reuse of used dentures and other dental structures on other persons may lead to infections and transmission of diseases.

**Storage conditions** Store in a cool, dry place away from sunlight. The liquid should be stored at a temperature <30°C.

### Disposal of the product

The powdered resin and polymerised product do not require specific disposal and can be disposed of in household waste. Monomer residues should be disposed of as chemical waste, do not pour into drains. Contaminated products with biological and/or infection hazards should be disposed of in accordance with applicable regulations.

### Hazard statements / Precautionary statements

**Liquid:** H225 Highly flammable liquid and vapour. H315 Causes skin irritation. H317 May cause an allergic skin reaction. H335 May cause respiratory irritation / P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P261 Avoid breathing dust/fume/gas/mist/vapours/spray. P262 Do not get in eyes, on skin, or on clothing. P280 Wear protective gloves/protective clothing/eye protection/face protection/hearing protection... P501 Dispose of container and/or any remaining contents in accordance with applicable regulations. **Powder:** EUH208: Contains (Benzoyl peroxide). May produce an allergic reaction. EUH210: Safety data sheet available on request.

### Information for the patient

The professional should inform the end-user of the following points:

- Maintain normal oral hygiene and adopt a daily hygiene protocol\*, avoid excessive brushing which may cause abrasive wear, and do not immerse the prosthesis in alkalis, acids or other substances that may damage the acrylic materials.
- At least once a week the denture should be removed and kept for cleaning overnight.

c) The material is not visible to X-rays.

d) The material is relatively tough, but strong bites should be avoided.

e) In case of allergic reactions to the material, consult a doctor.

f) Of the following residual risks: Allergies and irritations caused by the material. Breakage and possible ingestion of the product. Deterioration of the product. Incorrect use of the product. Inadequate hygiene and maintenance of the product.

**\*Hygiene protocol:** patients should brush the prosthesis at least once a day; the denture should be removed and gently cleaned. Soak and brush with a soft bristle brush and a non-abrasive denture cleanser to remove food debris, plaque and other deposits.

### Manufacturer's responsibility

Material for dental use: The manufacturer is not responsible for any damage that may result from use other than in accordance with the instructions for use. Class I medical device: Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the incident occurred.

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Unión Dental, S.A.